



MEDIHONEY® Paste

Wound and Burn Dressing

INSTRUCTIONS FOR USE



Sterilized using irradiation




Do not re-use



Not made with natural rubber latex



Catalogue Number

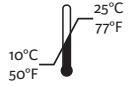

Temperature limit

Caution: Federal law restricts this device to sale by or on the order of a physician.



Do not re-sterilize



Manufacturer



Do not use if package is damaged

PRODUCT DESCRIPTION

MediHoney® Paste with Active *Leptospermum* Honey is a moist dressing made of 100% Active *Leptospermum* medical grade Honey. The dressing helps maintain a moist environment conducive to wound healing and helps aid and support autolytic debridement. The dressing can be removed without causing trauma to the wound when a primary dressing is used.

INDICATIONS

MediHoney Paste dressing with Active *Leptospermum* Honey provides a moist environment conducive to wound healing and is indicated for light to moderately exuding wounds such as:

- Diabetic foot ulcers
- Leg ulcers (venous stasis ulcers, arterial ulcers and leg ulcers of mixed etiology)
- Pressure ulcers / sores (partial and full thickness)
- 1st and 2nd degree burns
- Donor sites, traumatic and surgical wounds

CONTRAINDICATIONS

- 3rd degree burns
- Patients with a known sensitivity to honey
- Control of heavy bleeding

PRECAUTIONS

- Due to the dressing's low pH, some patients may notice slight transient stinging upon application of the dressing. If stinging persists and cannot be managed with an analgesic, remove dressing, cleanse area, and discontinue use.
- During initial use of the dressing (depending on wound exudate levels, interstitial fluid, and edema surrounding the wound), the dressing's high osmolarity may contribute to increased exudate. This can be managed by adjusting the frequency of dressing changes according to the level of exudate.
- During the healing process, due to autolytic debridement, it is common for non-viable tissue to be removed from the wound, resulting in an initial increase in wound size. Although an initial increase in size may be attributed to the normal removal of non-viable tissue, consult a healthcare professional if the wound continues to grow larger after the first few dressing changes.
- After removing the dressing, if some material remains and is not easily removed, soak with sterile saline or water until it is removed without difficulty.
- Consult a healthcare professional if you see signs of infection, which can include increased redness, increased pain, increased wound drainage, increased skin irritation, and changes in wound color and/or odor.
- Do not use this dressing with any other primary dressings, other than those listed in this package insert, without first consulting a healthcare professional.

DIRECTIONS FOR USE

- Prior to application of MediHoney Dressing, cleanse the wound area as necessary.
- For flip top cap, remove the cap from the tube and remove the sterile seal. For twist-off cap, twist and break off end of the nozzle to open tube.
- Apply either directly to the wound bed, or onto a suitable primary dressing such as alginate, gauze pad, or island dressing.
- Cover with an appropriate secondary dressing.

DRESSING CHANGE AND REMOVAL

- Dressing change frequency will depend on the condition of the patient as well as the level of wound exudate.
- Reapply MediHoney Paste when the secondary dressing has reached its absorbent capacity or as directed by a wound care professional.
- Moisten with sterile saline if the wound bed appears dry before removal of the dressing.
- Remove the secondary dressing from the wound bed gently.
- Cleanse the wound bed prior to application of new dressing.

If further information or guidance is required, please contact Integra LifeSciences Corporation for assistance.

www.integralife.com

1 (800) 654-2873 / (609) 275-0500 (USA)

1 (800) 263-6693 / (905) 632-7888 (Canada)

U.S. Patent integralife.com/patentmarking



Manufacturer:

Derma Sciences, Inc.

104 Shorting Road

Toronto, Ontario M1S 3S4

integralife.com

Made in Canada.

